



## Download & User Instructions For Android Devices

You can download Corventive Patient for your Android device by going to:

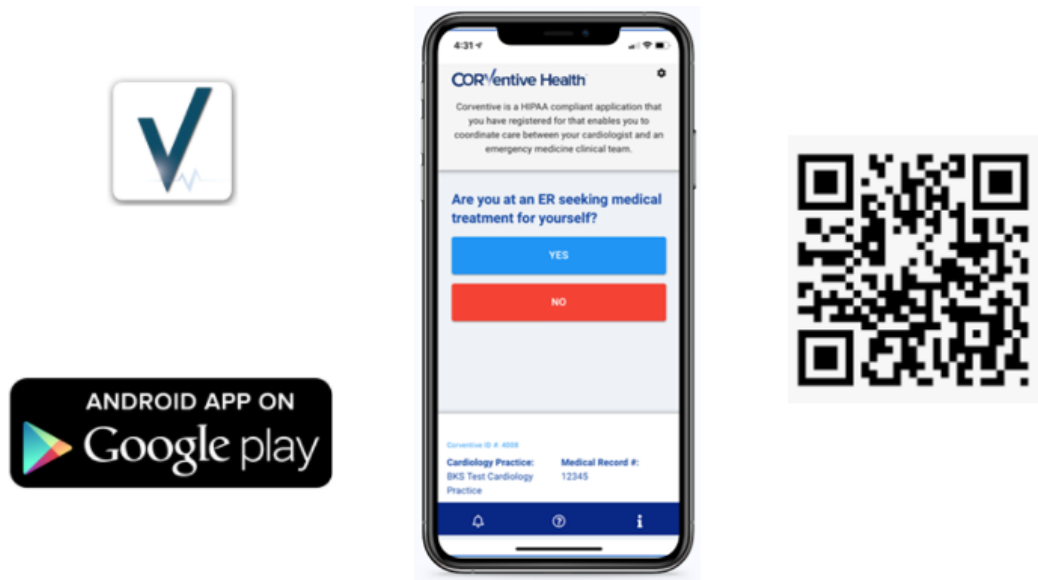

Please note, Corventive Health offers a patient and cardiologist version of our app. Please pay special attention to **avoid downloading Corventive Cardiologist** version of our app.

1. Download Corventive Patient.
2. When prompted "Allow all the time".

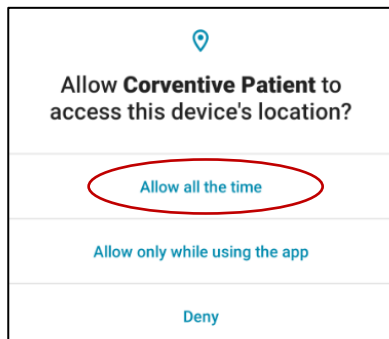

3. Please enter your mobile phone number.
  4. You will receive a 6-digit security code via SMS text message.
  5. Enter the 6-digit security code into Corventive Patient to verify your identity and authenticate the mobile application (No further sign-up or passwords are required).
  6. Agree to Corventive's Terms of Use.
  7. Close the Corventive Patient App.
- ❖ **Optional - You can disable location services if you desire to use Corventive Patient in manual mode only. Please note, we recommend you enable location services so Corventive Patient automatically activates in a hectic cardiac emergency situation.**